

# Nimbl™ Rechargeable Battery Quick Start Guide


## CHARGE YOUR BATTERY

Plug the Nimbl power supply into the wall and into the port on the battery. Your battery will be fully charged when the LEDs stop flashing. Once charged, disconnect the battery from the power supply.

## BEGIN USING YOUR BATTERY

Plug the battery into the power port on your controller to operate the controller.

## CHECK YOUR BATTERY'S CHARGE LEVEL

Press the  button on the front of the battery to confirm the charge level. Five illuminated LEDs indicate a full charge and zero illuminated LEDs indicate no charge. A full charge lasts about 2.5 hours.



nimbl™



Scan to access the Rechargeable Battery User Guide

**Tactile Medical**

3701 Wayzata Blvd, Suite 300  
Minneapolis, MN 55416 USA

[tactilemedical.com](https://tactilemedical.com)

**Customer Care**

**Text or Call: 612.355.5100**

Toll Free Phone: 833.3TACTILE (833.382.2845)

Fax: 612.355.5101 / Toll Free Fax: 866.435.3949

Email: [customerservice@tactilemedical.com](mailto:customerservice@tactilemedical.com)

Hours: 7 a.m. to 7 p.m. CT, Monday–Friday

**Tactile**  
**MEDICAL**<sup>®</sup>  
HEALING RIGHT AT HOME

Tactile Medical is a registered trademark and Nimbl and Healing Right at Home are trademarks of Tactile Medical.

©2024 Tactile Medical. All rights reserved. 704178-001 Rev A 8/2024